PRODUCT DESCRIPTION

Forendo Paste is a temporary or permanent endodontic filling material indicated for routine use as a root canal dressing between office visits; for treatment of abscesses, periapical lesions, perforations, exudation, root resorption, traumatic injuries; and for apexification and apexogenesis. Forendo Paste can be used alone as a temporary canal treatment for vital and non-vital pulpotomies, and for pulpectomies in deciduous teeth. Forendo Paste is a non-setting, premixed paste that contains calcium hydroxide and iodoform in a silicone oil base. It has a pH>12 and is radiopaque.

INDICATIONS

Root canal therapy

CONTRAINDICATIONS

Not a hard-setting root canal filling material

INSTRUCTIONS FOR USE

TEMPORARY CANAL TREATMENT

- Using good endodontic procedures, cleanse and prepare the root canal in the conventional manner. Dry the canal.
- Place a new disposable applicator tip securely on the luer lock syringe.
- Fill the canal by applying gentle pressure on the syringe plunger. Do not force the plunger. The paste can also be placed into the canal with a lentulo spiral.
- Remove excess at the orifice with a cotton pellet.
- Seal access opening with your preferred cement.
- Discard applicator tip and cap syringe immediately after use.
- Forendo Paste is easily removed with file and irrigation with sodium hypochlorite.

TREATMENT OF PRIMARY TEETH

- Follow instructions above for Temporary Canal Treatment.
- It is not necessary to remove Forendo Paste from the root canal. The material will resorb with the roots in deciduous teeth.

STORAGE AND HANDLING

- When dispensing from the syringe, do not force plunger. Apply light pressure only. If material does not dispense, remove needle, purge some material from the syringe, and place a new needle on the syringe.
- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap syringe immediately after use.

CAUTION

For professional use only by dentists. Wear protective eyewear and surgical gloves. Avoid contact with eyes, mucous membranes and skin. If contact occurs, flush with copious amount of water. If contact occurs with eyes, flush with copious amounts of water and seek medical attention immediately.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multi-ple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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SAFETY DATA SHEET

A square QR code with the text "PULPDENT®" centered below it. The entire QR code and text are enclosed within a black rectangular border.

A black rounded rectangular button. On the left is a white circular icon containing a black smartphone silhouette. To the right of the icon, the text "SCAN ME" is written in white, uppercase letters.

www.pulpdent.com/info

A black and white icon of a factory with three smokestacks. Below the factory, the letters "USA" are written in a bold, sans-serif font.

A red thermometer icon. A red line extends from the top of the thermometer, with "27° C" at the end. Another red line extends from the bottom of the thermometer, with "5° C" at the end.

A black icon of a flame with a sun-like symbol in the center. Below the flame is a solid black square.

A red diamond-shaped warning sign with a black exclamation mark in the center.

PULPDENT®

www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.

(617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262